

Dam It

Instructions

Dam It eliminates gagging & assures capturing 2nd molars! Dam It membrane creates a flexible barrier that prevents material from running down the throat. It maintains the hydraulic pressure against the distal of second molars.

Dam It membrane is pulled from the dispenser until the repeat mark shows (the solid double lines running across the width of the membranes). One repeat is used per tray side.

Dam It incorporates into the impression material. It can even become folded into a margin without creating a dimensional error.



91738 Rev A

- ① Push the cutter down and pull up on the membrane to tear off a double section to accommodate both sides.




- ② Align the dotted red line with the bottom of the tray.




- ③ Wrap the self adhesive membrane around the posterior of each side of the tray.



- ④ Wrap the membrane under the tray.

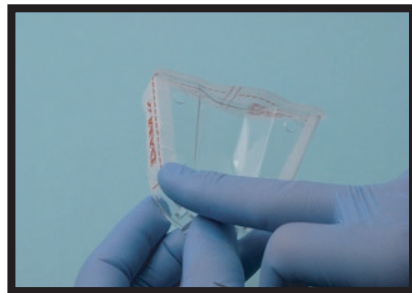

- ⑤ With scissors, cut three small slits halfway down the posterior loop.

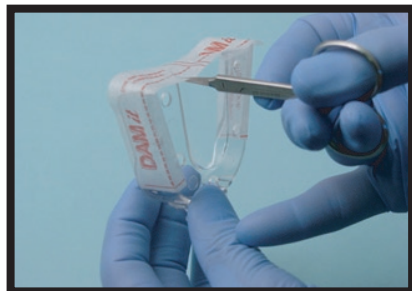

- ⑥ Fill the tray and place it. For best results, use the 5th Hand retractor!

