

*The  
ideal  
material  
for every  
technique  
and  
speed!*

### Triple Tray\*

Light Body is syringed around teeth and gingiva. The patient closes on a tray filled with Heavy Flex.

Use:	<b>First Quarter</b> Light Body & Monophase or Heavy Flex	<i>1<sup>3</sup>/<sub>4</sub>min</i>
	<b>First Half</b> Light Body & Heavy Flex	<i>2<sup>1</sup>/<sub>4</sub>min</i>
	<b>Star VPS</b> Light Body & Heavy Flex	<i>4<sup>1</sup>/<sub>2</sub>min</i>

### Dr. John Kois, full arch

Fill tray with Ultra Heavy injectable putty and let stand. Syringe Ultra Light around teeth and on top of tray. Place tray until set.

Use:	<b>Star VPS</b> Ultra Heavy & Ultra Light	<i>4<sup>1</sup>/<sub>2</sub>min</i>
------	---	--------------------------------------

### H & H, Dr. Jeff Hoos

Patient closes on Stiff Bite in Triple Tray. Retain impression on opposing arch. A fast set wash material is placed in the impressions. The patient closes again, forcing the material hydraulically, under the gingiva, without cord, expelling fluids!

Use:	<b>First Quarter</b> Light Body	<i>1<sup>3</sup>/<sub>4</sub>min</i>
	<b>First Half</b> Light Body	<i>2<sup>1</sup>/<sub>4</sub>min</i>
	<b>Star VPS</b> Heavy Stiff Bite	<i>50sec</i>

### Putty/Tray, full arch

Putty is placed in tray and plastic wrap is placed on top. Place tray until partially set and then remove from mouth. Plastic wrap is removed after the putty is fully set. Light body material is syringed around the teeth and into tray, then the tray is placed in the mouth.

Use:	<b>Star VPS</b> Putty & Light Body	<i>4<sup>1</sup>/<sub>2</sub>min</i>
------	------------------------------------	--------------------------------------

### Temporary Veneers & Recreating Bite Anatomy

Syringe Clear Bite over tooth before prep (no tray). **Veneer:** After prep, syringe composite in impression and place. Squeeze out excess & light-cure through impression material. **Occlusal:** Place impression over composite, have patient close and cure through edges then through occlusal.

Use:	<b>Star VPS</b> Clear Bite	<i>2<sup>1</sup>/<sub>4</sub>min</i>
------	----------------------------	--------------------------------------

### Full Arch, two viscosity, simultaneous set

Light Body is syringed around teeth and gingiva. Place tray filled with Heavy Flex.

Use:	<b>Star VPS</b> Light Body & Heavy Flex	<i>4<sup>1</sup>/<sub>2</sub>min</i>
------	---	--------------------------------------

### Provisional Impressions, single viscosity

Place quadrant tray filled with Monophase until set.

Use:	<b>First Quarter</b> Monophase	<i>1<sup>3</sup>/<sub>4</sub>min</i>
------	--------------------------------	--------------------------------------

### Orthodontic, full arch

Fill tray and place tray until set.

Use:	<b>First Half</b> Monophase Ortho	<i>2<sup>1</sup>/<sub>4</sub>min</i>
	<b>First Half</b> Monophase Ortho PM (for Pentamix)	<i>2<sup>1</sup>/<sub>4</sub>min</i>

\*Triple Tray is not a Danville trademark.

