

clean the unit insure all power is disconnected to the unit. Over a trash receptacle unlatch the rear door, remove the filter element and shake the abrasive of the cabinet. Clean the filter by tapping it against the inside of the trash receptacle. The filter should not be washed. Filters should be replaced if they become torn or perforated or if airflow remains significantly impaired even after cleaning. There are many types and sizes of filters these may not provide proper sealing of filtering of abrasive. Replacement filters may be purchased from Danville Materials ordering P/N 116545. Average filter life is between one and two years.

Replacement of light

The fluorescent lamp in the cabinet is a Sylvania Dulux 9 watt bulb and may be purchased from either a home improvement store or from Danville Materials P/N 59090. The bulb is pulled from its socket for replacement. **Use care not to apply excessive pressure on the bulb as it can shatter and cut fingers.**

Window Replacement

The window will become frosted in time from use, it is easily replaced. To prevent premature scratching, we advise the following cleaning procedures: roll a damp cloth over the surface. It is essential that a clean surface on the cloth always be exposed to prevent abrasive particles from being dragged across the surface of the window. If it becomes necessary to replace your window you may order a replacement from Danville Materials P/N 185508.



Warranty

The Macrocab is guaranteed to be free of defects in material and workmanship for a period of one year. Normal wear and tear and abuse are excluded. Warranty is limited to repair or replacement. No other warranty is expressed or implied. In all cases, original jurisdiction shall rest in Contra Costa County, California, U.S.A.

Cleaning

The interior of the dust cabinet should be cleaned periodically. Excessive accumulation of abrasive dust will plug the filter and reduce dust-filtering efficiency. To

DANVILLE MATERIALS
3420 Fosteria Way Ste. A-200
San Ramon, CA 94583 USA
Ph: (800) 827-7940
Fax: (925) 973-0764

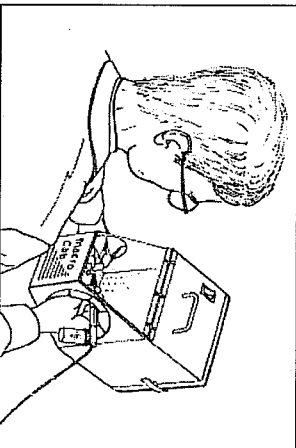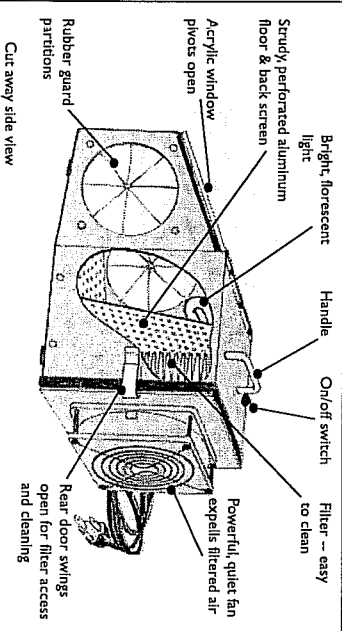

clean the unit insure all power is disconnected to the unit. Over a trash receptacle unlatch the rear door, remove the filter element and shake the abrasive of the cabinet. Clean the filter by tapping it against the inside of the trash receptacle. The filter should not be washed. Filters should be replaced if they become torn or perforated or if airflow remains significantly impaired even after cleaning. There are many types and sizes of filters these may not provide proper sealing of filtering of abrasive. Replacement filters may be purchased from Danville Materials ordering P/N 116545. Average filter life is between one and two years.

Replacement of light

The fluorescent lamp in the cabinet is a Sylvania Dulux 9 watt bulb and may be purchased from either a home improvement store or from Danville Materials P/N 59090. The bulb is pulled from its socket for replacement. **Use care not to apply excessive pressure on the bulb as it can shatter and cut fingers.**

Window Replacement

The window will become frosted in time from use, it is easily replaced. To prevent premature scratching, we advise the following cleaning procedures: roll a damp cloth over the surface. It is essential that a clean surface on the cloth always be exposed to prevent abrasive particles from being dragged across the surface of the window. If it becomes necessary to replace your window you may order a replacement from Danville Materials P/N 185508.

Warranty

The Macrocab is guaranteed to be free of defects in material and workmanship for a period of one year. Normal wear and tear and abuse are excluded. Warranty is limited to repair or replacement. No other warranty is expressed or implied. In all cases, original jurisdiction shall rest in Contra Costa County, California, U.S.A.

Cleaning

The interior of the dust cabinet should be cleaned periodically. Excessive accumulation of abrasive dust will plug the filter and reduce dust-filtering efficiency. To

DANVILLE MATERIALS
3420 Fosteria Way Ste. A-200
San Ramon, CA 94583 USA
Ph: (800) 827-7940
Fax: (925) 973-0764

